ENGLISH

Directions for use:

1. Wash your hands.
2. Clean and dry anal area well.
3. Remove foil.
4. Release thread from the tampon.
5. Soften the tampon in lukewarm water (30 seconds).
6. Squeeze out the water from the tampon.
7. Place (reusable) applicator in the opening at the bottom of the tampon.
8. Push the tampon past the orbicular muscle, only the thread remains visible.
9. Hold the tampon with the other index finger while withdrawing the applicator. Then push the tampon further inside with your finger.
10. Fasten the thread with sticking plaster if necessary.
11. Wash your hands.
12. The tampon can easily be removed by means of the thread. Never wash a tampon through the toilet.

The tampon can be inserted while lying on one's back or side. Compare this procedure to inserting a suppository.

Tips and advice:

1. Do not use the same tampon for more than 12 hours. Build up the duration of use gradually.
2. Before inserting a new tampon, allow some time to empty the bowels.
3. Relax while inserting a tampon.
4. Resistance and pain may occur during first use. Do not force anything.
5. See to good toilet training by going to the toilet at fixed times.
6. Vaseline on the tampon and the anus makes inserting easier. A large applicator will help with a poor hand function.

Contraindications: infections, diarrhoea, after radiotherapy.

• A-TAM is a disposable product and must not be reused. It could lead to bacterial contamination.

In case a serious incident has occurred in relation to the device: Inform Med SSE as listed on the

packaging, your local distributor and your local healthcare authority.
Do not use tampon if you are sensitive to PVA-foam products. No known side effects.

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MED SSE System GmbH
Medeco B.V.