klinispon nose plug

KLINION®

HemCon Medical Technologies CZ s.r.o.
Za Mlýnem 5
CZ 66601 Třnov
Czech Republic

Distributed by
Medeco B.V.
Alexander Flemingstraat 2
NL-3261 MA Oud-Beijerland
The Netherlands

www.medeco.org

IFU_KWC060_02 / 24-09-2020 /
NP011B Rev 5 / 0_040103

ENGLISH

Product Description

Expandable PVA sponge impregnated with M-doc™ (micro-dispersed oxidized cellulose), which helps to terminate bleeding. Klinispon nose plugs absorb blood and expand into a soft sponge. The nasal plugs do not stick to the walls of the nose, allowing easy removal and reducing the risk of renewed bleeding.

Medical device.

Indications

Expandable dressing for nosebleeds.

Directions for use

Remove the Klinispon nose plug from the packaging. When inserting into the nose, ensure a piece of the product remains outside the nose to allow easy removal. The nose plug will expand and soften like a sponge as it absorbs blood. After a few minutes, when bleeding has stopped, carefully remove the nose plug from the nose. Dispose according to local procedures and guidelines.
Note: It is not recommended to use for children under 5 years. Prior to use for children over 5 years, the plug can easily be cut to size using scissors.

Precautions

- Do not use on patients with a known sensitivity for one of the products components.
- Do not leave the plug in the nose for more than 1 hour.
- In the event of prolonged bleeding, consult a healthcare professional.
- Do not re-use. Re-use can cause serious infections.
- Only use 1 nose plug at a time.

In case a serious incident has occurred in relation to the device: Inform HemCon Medical Technologies CZ s.r.o. as listed on the packaging, your local distributor and your local healthcare authority.

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