klinisport® cold / hot pack & cold pack instant

INSTRUCTIONS FOR USE

ENGLISH

Product Description Cold/Hot Pack
Cold/hot compresses are for external use only, suitable for use, by untrained non-specialists both in medical institutions and in domestic environments. Cold/hot compresses can be used in the following areas of application:

- Cold treatment: bruises, inflammation, insect bites, hangovers, headache, minor burns, migraine, nose bleeds, contusions, swelling, dislocation, sprains, tennis elbow, toothache, strained muscles.
- Hot treatment: lumbago, menstruation pain, aching muscles, backache, tension.

Cold/hot compresses must not be used if patients suffer from circulatory disorder and cryaesthesia, Raynaud syndrome. Patients suffering from cardiovascular disease and diabetes mellitus may only use the product following consultation with their doctor.

Directions for use Cold/Hot Pack

Use as a cold compress:
Cool the compress in the freezer compartment down to approx. -18°C for 2 hours before use. Wind a thin cloth around the compress for use. We recommend keeping the compress stored in the freezer compartment after use so that it is always ready for re-use.
Use as a hot compress:
Before use place the compress in hot water up to +80°C or in the microwave at 280 W (under constant observation). After the heating up process, take the compress out and knead it well to distribute the heat. Wind a thin cloth around the compress for use.
Caution! Overheating can destroy the compresses. Leaking or emptied compresses must not be heated and re-used.

Precautions Cold/Hot Pack

The gel used in the compresses is not toxic. Should any gel get onto the skin or into your eyes, rinse off or out using plenty of water. Consult a doctor if necessary.

Keep the cold/hot compresses in a cool and dry place protected from sunlight. After application the cold/hot compress can be cleaned with standard washing-up liquid and water if necessary. Damaged, leaking compresses must not be re-used again and must be disposed of with household waste. Keep cold/hot compresses out of the reach of children and animals.

The single use cold pack instant cannot be heated! The heating-up time for the reusable cold/hot compress depends on the size of the compress and is as follows:		
Size of the compress	Water bath	Microwave
10 x 12 cm	+/- 2 min.	+/- 45 sec.
12 x 29 cm	+/- 6 min.	+/- 4 min.

Product Description Cold Pack Instant

Instant cold packs are for single use only and can be used for immediate treatment in the following areas of application: sports injuries, physiotherapy, sports medicine, orthopedics, gynaecology, dental medicine and post-operative therapy.

Directions for use Cold Pack Instant

For rapid cooling of blunt injuries, to prevent and reduce swelling and for pain relief.
Locate the internal fluid bag and squeeze it firmly until it pops. Shake the bag thoroughly. For external use only.

Precautions Cold Pack Instant

- Do not apply on open wounds.
- Do not use for more than 20 minutes.
- Always wrap in a thin cloth.
- Keep out of reach of children and animals.
- The compresses should not be used on patients with circulatory disorders, hypersensitivity to cold and Raynaud's syndrome.
- Use in cardiovascular diseases and diabetes mellitus only after consultation with a doctor.

If the solution comes into contact with eyes, skin or open wounds, immediately rinse with plenty of water and if in doubt, consult a doctor or pharmacist.

Store in a cool and dry place, not below 0°C or above 45°C and away from direct sunlight.

Please report any serious incidents which happen in connection with the product to Gello GmbH Geltechnik and the authorities responsible.

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